



PRODOLPHIN™

Surgical Technique Guide
TLIF PEEK Cage



www.prodorth.com



contents

02
04

Surgical Technique of Prodorth
TLIF PEEK Cage

Intended Use of the Device

Secondary and Possible Side
Effects

05
06

PRODOLPHIN™ - Size

Prodorth TLIF PEEK Cage
Instruments

07

Step 1
Patient Positioning and Exposure

Step 2
Preparation of the Disc Spaces

Step 3
Insertion of the Trial Implants

08
09

Step 4
Connection of the TLIF
PEEK Cage with its Inserter

Step 5
Releasing the TLIF PEEK
Cage from its Inserter



PRODOLPHIN™

SURGICAL TECHNIQUE OF PRODORTH TLIF PEEK CAGE

Prodorth has developed this surgical technique document for surgeons and healthcare professionals but not for unauthorized persons. This document is a supportive source but not a complete instruction for an inexperienced surgeon to perform the entire surgery, therefore the information within this surgical technique should be considered with the previous medical experiences and education of the surgeon. Surgeon's medical judgement and decisions will be the best treatment for the patient and the results will be different according to the patient's physical and mental situation.

DEVICE DESCRIPTION

Prodorth TLIF PEEK Cage is a long-term implant in order to dispose of the patients' complaints which are raised because of the pain arising from the herniation at the lumbar discs, and traumas on the lumbar spine.

Prodorth TLIF PEEK Cages are long-term implants, however, they are not able to withstand the forces like healthy bone structures.

Current Status of the Device: Device is already CE marked (since 2013) and has been on the market.

TLIF PEEK Cage GMDN No: 60762

Product Class: (Annex II of Directive 93/42/EEC) Class IIb

Raw Materials: Ti6Al4V-ELI (ASTM F 136 / ISO 5832-3) and PEEK (ASTM F 2026)

Biological Assessment:

Biological Assessment of Device According to TS EN ISO 10993-1 : 2021

| | |
|------------------|---------------------------|
| Category | Implant Device |
| Contact Level | Bone / Tissue |
| Contact Duration | C (Permanent - > 30 days) |

STERILIZATION

Prodorth TLIF PEEK Cage is released to market as non-sterile.

They must be sterilized prior to surgical use. All packaging materials are removed prior to sterilization. The recommended sterilization method for Prodorth cages is steam sterilization in an autoclave. The products which are intended to be sterilized should remain in an autoclave at 134 °C for 18 minutes. There is no other sterilization method Prodorth recommends.

INTENDED PURPOSE OF THE DEVICE

Prodorth TLIF PEEK Cage is a long-term implant in order to dispose of the patients' complaints which are raised because of the pain arising from the herniation at the lumbar discs, and traumas on the lumbar spine.

- It is a single-use device
- Does not include human or animal tissue and phthalate
- Does not include any software or accessory
- The product is supplied as non-sterile
- Product does not cause any radioactive source or beam diffusing

Population: Skeletally mature male / female patients

Intended User(s): Healthcare professionals (Surgeons trained and experienced in the related field.)

INDICATIONS

General criteria and principles related to instrumented spinal surgery are applied here:

- Degenerative disc pathologies
- Herniated nucleus pulposus
- Grade 1 degenerative or isthmic spondylosis
- Visible loss of disc height compared to adjacent levels
- Lumbar pseudarthrosis

Note: Patients should be skeletally mature and have had six months of non-operative treatment.

CONTRAINDICATIONS

Prodorth TLIF PEEK Cages should never be used at any condition not described in the indications for use. Contraindications include, but are not limited to:

- Fracture, tumor
- Any abnormality present which affects the normal process of bone remodeling including, but not limited to, severe osteoporosis involving the spine, bone absorption, osteopenia
- Marked local inflammation
- Osteoporosis, calcium metabolism disorder
- Pregnancy
- Infection
- Recognized allergies to titanium or titanium alloys and PEEK material
- Damaged lumbar vertebrae from an accident (trauma) at the level of the surgery
- Prior fusion at the level(s) to be treated
- An unhealthy shape (deformity) of the lumbar vertebrae at the level of the surgery
- Low bone mineral density, such as osteoporosis or osteopenia
- Mental disability
- Obesity
- Open wounds
- Fever or leukocytosis
- Alcohol or drug addiction
- Uncooperative patient or patient with neurologic disorders rendering the patient incapable of following instructions

These contraindications can be relative or absolute and should be considered when physician makes a decision. The above list does not include all possibilities. Surgeons should discuss relative contraindications with the patient.

SECONDARY AND POSSIBLE SIDE EFFECTS

The patient shall be notified regarding the below mentioned adverse events pre-operatively. A second surgical treatment may be required:

- Pseudarthrosis
- Implant penetration, migration or implant failure
- Infection
- Tissue or nerve damage caused by improper positioning and placement of implants or instruments
- Paralysis
- Allergy to materials used
- Loosening
- Increased pain
- Instability
- Hematoma
- Pain or illness
- Nonunion or delayed union of the bone
- Bleeding blood vessels
- Bursitis
- Inability to perform daily activities
- Dura leak requiring a repeating surgery
- Intervertebral cages can be fractured postoperatively above or below segments of the surgical level due to trauma, the presence of any defect or weak bone structure. Re-operation may be required
- Wound infection
- Displacement of the disc adjacent segment degeneration
- Death

WARNINGS

- Never re-use an implant even in a perfect state. Any implant which has been used, twisted, bent, implanted and then removed even if it appears intact, it must be discarded
- Use new implants routinely
- Similar products of competitors shall not be combined with the components of the Prodorth TLIF PEEK Cage. Prodorth implants and instruments should only be used with Prodorth instruments. In case of using other company's instruments, this might result in galvanic corrosion, incompatibility between the products as well
- No component of the Prodorth TLIF PEEK Cage shall be reused
- The restricted shelf life of the device is 10 years. It should never be used after its expiration date
- Correct selection of the implant is highly important!
- Use of provided trials is recommended

PRODORTH TLIF PEEK CAGE DESCRIPTION

Prodorth TLIF PEEK Cage is intended for single use only and restores degenerative disc pathologies. The unstable situation of the lumbar spine causes herniation of discs and this makes pressure on nerves. In order to solve this problem, TLIF PEEK Cage is positioned at the intervertebral area after discectomy, so that 2 vertebrae work as one vertebra as a result of bone fusion. In this way, the pressure on nerves is disposed and the patient has relief.

Prodorth TLIF PEEK Cage is made of PEEK (Polyether-ether-ketone/ ASTM F2026) which is a polymer-based composite material and Ti6Al4V –ELI (Grade23) material.


X-ray marker pins for visibility.

The TLIF PEEK Cage is anatomically-shaped to fit the lumbar disc space optimally, with two footprint sizes and multiple heights to restore disc height. A large bone graft window accommodates bone grafts or synthetic bone inserts.

TLIF PEEK Cages provide uninterrupted guidance during operation. Circular toothed surface feature for holding on to the inferior and superior endplates.




It easily adapts to the required position

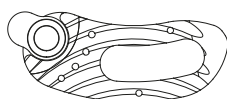

25

| SIZE | REF.CODE |
|----------|---------------|
| 25x7 mm | 102.05 002507 |
| 25x8 mm | 102.05 002508 |
| 25x9 mm | 102.05 002509 |
| 25x10 mm | 102.05 002510 |
| 25x11mm | 102.05 002511 |
| 25x12 mm | 102.05 002512 |
| 25x13 mm | 102.05 002513 |
| 25x14 mm | 102.05 002514 |
| 25x15 mm | 102.05 002515 |

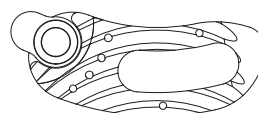

28

| SIZE | REF.CODE |
|----------|---------------|
| 28x7 mm | 102.05 002807 |
| 28x8 mm | 102.05 002808 |
| 28x9 mm | 102.05 002809 |
| 28x10 mm | 102.05 002810 |
| 28x11mm | 102.05 002811 |
| 28x12 mm | 102.05 002812 |
| 28x13 mm | 102.05 002813 |
| 28x14 mm | 102.05 002814 |
| 28x15 mm | 102.05 002815 |



PRODORTH TLIF PEEK CAGE INSTRUMENTS

Prodorth offers different designs of instruments for each step of the surgical procedure. They have been designed as simply as possible and user-friendly in order to provide ease of use.

Prodorth instruments are made of stainless chrome nickel steel, aluminum, and silicone.



▶▶▶ TLIF Inserter PL 200.20.001



Mallet PL 200.20.008 ◀◀◀



▶▶▶ Left Angled Hole Curette PL 200.20.003



Trial Implant Inserter PL 200.20.009 ◀◀◀



▶▶▶ Straight Hole Curette PL 200.20.004



▶▶▶ Right Angled Hole Curette PL 200.20.005



T Handle Locking 1/4" PL 200.20.011 ◀◀◀



▶▶▶ Reverse Cup Curette PL 200.20.006



Shaver 1/4" 7 mm PL 200.20.012 ◀◀◀
 Shaver 1/4" 8 mm PL 200.20.013
 Shaver 1/4" 9 mm PL 200.20.014
 Shaver 1/4" 10 mm PL 200.20.015
 Shaver 1/4" 11 mm PL 200.20.016
 Shaver 1/4" 12 mm PL 200.20.017



▶▶▶ Straight Cup Curette PL 200.20.007

SURGICAL PROCEDURE

Step 1 Patient Positioning and Exposure

The patient is positioned properly in the prone position. The general essentials of lumbar surgery are applied. (Figure 1)



Figure 1

Step 2 Preparation of the Disc Spaces

Use Prodorth Curettes for removing the disc from disc spaces. A proper discectomy is essential for the success of the surgery. (Figure 2)



Figure 2

Step 3 Insertion of the Trial Implants

The trial implants are connected to the Trial Implant Inserter PL 200.20.009 and then introduced through the vertebrae in order to determine the accurate size of the implant. (Figure 3)



Figure 3

Step 4

Connection of the TLIF PEEK Cage with its Inserter

After selecting the appropriate size of the TLIF PEEK Cage, it is connected to the TLIF Inserter (PL 200.20.001). In order to connect it accurately, the knob behind the inserter is rotated clockwise until assuring the implant is completely connected. (Step 4.1)

After connecting with the inserter properly, the TLIF PEEK Cage can be introduced at the required angle. It can be introduced upright position. The introduction angle of the TLIF PEEK Cage can be adjusted by rotating the knob. (Step 4.2)

Step 4.1



Step 4.2



During the placement of the TLIF PEEK Cage, the knob should be tightened.

Step 4.3



Step 4.4



Step 4.5




As it's represented in figures 4.3, 4.4 and 4.5 the TLIF PEEK Cage is positioned as required while the knob is tightened and loosened.

Step 5 Releasing the TLIF PEEK Cage from its Inserter

Finally, the knob is loosened and the TLIF PEEK Cage is released at the required position. (Figure 5)



Figure 5

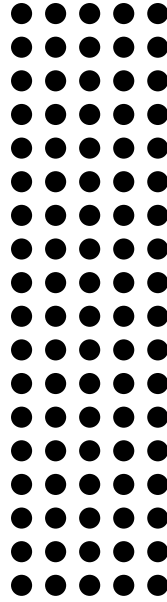

Surgical Technique Guide

TLIF PEEK CAGE



See the IFU prior to use for additional information.



Please check our website for the latest version of this Surgical Technique.



2292



TD.02.06.06 / Rev.05 / 25.07.2022

info@prodorth.com



0090 2323 48 49 50 (pbx)



Karacaoğlan Mah. Bornova Cad.
Öztim İş Merkezi No:9/G/1
Bornova - İzmir / TURKEY

